

Instructions for configuring Safe•t mate Model SM2 to prevent chair rollback while user is seated

CAUTION

This configuration will allow forward movement of the wheelchair with the user seated while preventing rearward mobility. Rearward movement of the chair can be achieved only by the caregiver by means of the temporary override lever. Please be advised that if the user rolls up to an immovable object such as a wall, desk, bed, window etc. **they will not be able to back-up**. For this reason, this configuration should only be used in a closely supervised situation.

The use of rear anti-tippers (Model SM-008) is strongly recommended in combination with this configuration.

1. Install the anti-rollback system completely per instructions.
2. Remove seat lever assembly (consisting of lever and articulating seat pad). Retain 2 bolts and nuts.
3. Temporary override lever must remain installed in order for caregiver to move chair backward.

