

Safe-t mate[®] Under-seat Fall Monitoring System

SM-009
Patent Pending

1. Loosen two back screws on both sides of wheelchair seat. (Figure 1)

Note: Perform steps 2-7 from behind wheelchair.

2. With under-seat unit beneath seat upholstery, slide straps between frame and upholstery forward of rear screws. (Figure 1 and 2)

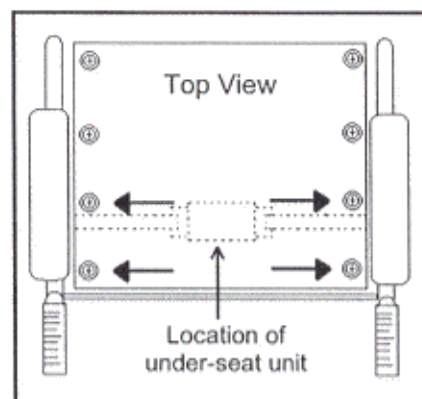


Figure 1

3. Slide straps as close to forward loosened screws as possible and retighten all four screws.

4. Loop each strap through ring on opposite end. (Figure 2)

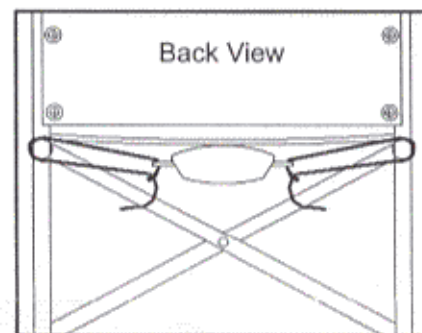


Figure 2

5. With under-seat unit centered left to right, pull ends of straps outward until taught and mate hook and loop fasteners. (Figure 3)



Figure 3

6. Connect alarm box. Clip allows mounting to seat back or chair frame. Route cord away from moving parts.

Note: Seat must be occupied for final adjustment.

7. **If a seat cushion is used, install it before final adjustment.** With the seat unoccupied, switch on the alarm and the horn should sound. Have someone sit on the seat and the horn should silence. If the under-seat unit causes an uncomfortable pressure on the occupant, release both hook and loop fasteners and loosen slightly. While seat is still occupied, adjust straps until you arrive at a position where alarm is silenced and seating is comfortable.



Figure 4

8. Install hook and loop safety clips to prevent accidental release of straps. (Figure 4)

CAUTION

This device is not a substitute for proper Supervision. Check fall risk patients frequently. Change battery monthly.

Note: Disconnect and remove alarm box prior to wheelchair Cleaning.

Manufactured by:
210 Innovations, LLC
46 Taugwonk Spur
Stonington, CT 06378