

# Installation Instructions

## Tools required:

1/4" wrench, 3/8" wrench

### 1. Set Anti-Tipper Wheel Height


Temporarily place and hold an anti-tipper bracket in the illustrated mounting position. Select height adjustment of wheel bracket that is closest to **but not contacting** floor. Install left and right wheel brackets using 1/4" Carriage bolts, washers and nuts provided.

### 2. Install Anti-Tipper Brackets

**Anti-Tipper Brackets must be installed on the outer side of the frame and front wheel posts as shown.**

Install two U-Bolts around chair frame and guide mounting bracket into position. Install 4 washers and nuts loosely. Spin front caster so that it is in the forward position as shown. Slide mounting brackets forward or rearward as needed to make sure anti-tipper assembly clears the front caster by approximately 1". Hold assembly with wheel bracket in vertical position, **securely tighten nuts** using 3/8" wrench.

**Periodically check for proper adjustment and tightness.**

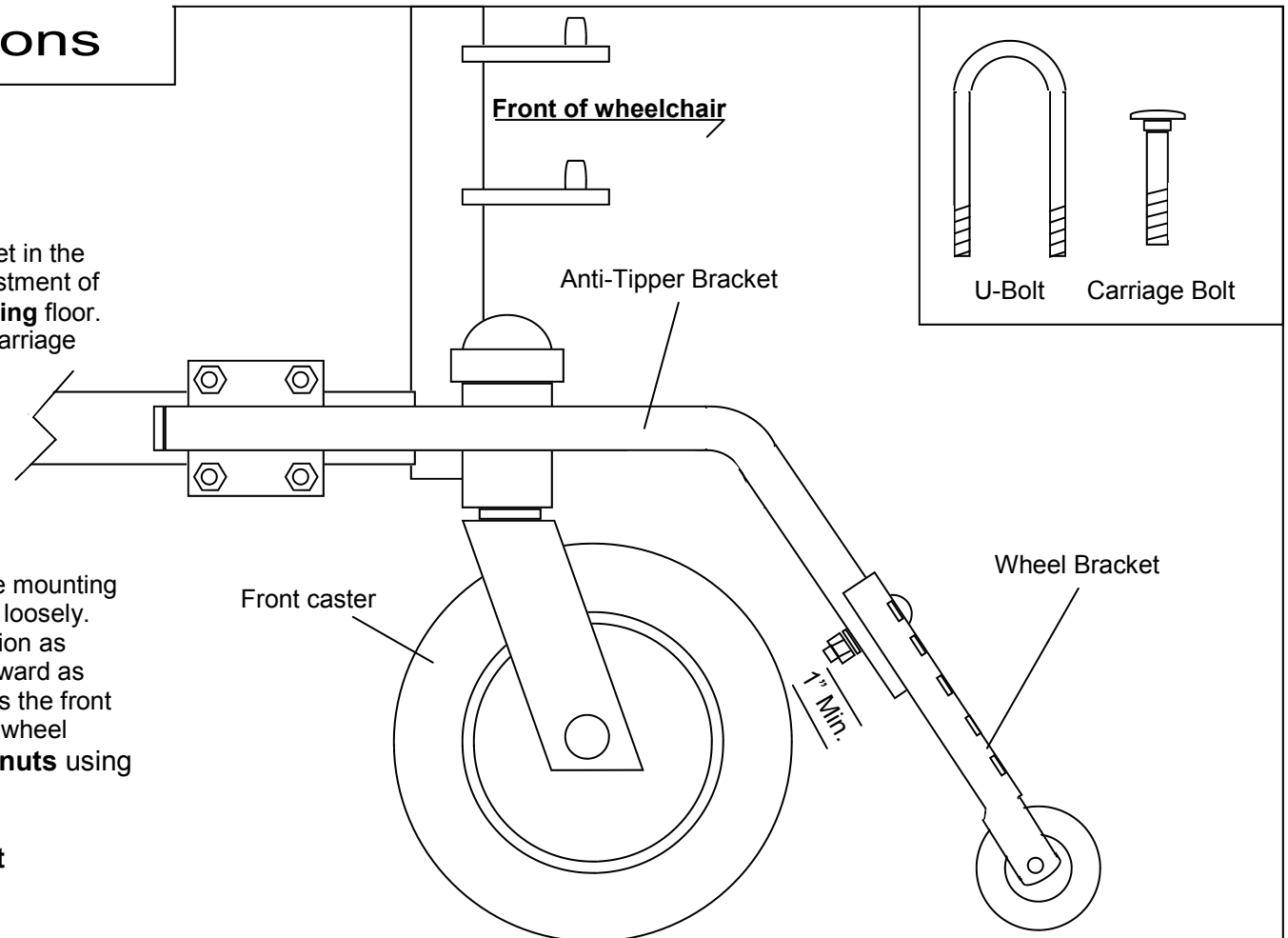

***Safe•mate***<sup>®</sup>

Model No. SM-011

Adjustable Wheelchair Front Anti-Tippers

Manufactured by  
210 Innovations, LLC  
34 Taugwonk Spur  
Stonington, CT 06378

[www.safetmate.com](http://www.safetmate.com)