

1. Remove existing sling seat. Two of the screws and washers will be reused for this installation.

2. Sticker under seat indicates back



**Refer to Illustration A.**

Loop webbing straps around both seat frame and main frame on each side of the wheelchair.

3. **Refer to Illustration B.**

The strap **MUST** be inserted from above the buckle. Slide strap between top of buckle and adjusting cam so that end of strap is pulled outward toward chair side frame. Depress spring loaded cam lever upward to facilitate sliding strap.

4. **Refer to Illustration C.**

Once the desired seat height is obtained, install Cam Locks as illustrated with lock inserted between cam lever and buckle. Slide Retaining Pin into place and secure with spring clip. This step ensures cam lever can not be accidentally depressed. Use plastic clip to secure excess strap.

5. **Refer to Illustration D.**

Wrap pre-installed cinching strap around both buckles and beneath cross bars. Bring the end through the plastic ring and draw tight before connecting hook and loop fastener.

This step ensures seat will not tip upward.

6. **Refer to Illustration A.**

Place supplied curved washers over leading edge of front straps to prevent straps from shifting forward. Use original screws and finish washers installed in forward most seat upholstery screw holes.

7. Finally, this seat deck is meant to be used with a seat cushion. Test cushion for proper fit. It may be beneficial to use a non-slip material between the seat deck and cushion.

Check seat deck periodically for proper adjustment. Clean with mild detergent. Seat can remain installed during wheelchair washing.

