

Safe•t mate

Product use and specification sheet

| | |
|---------------------|--|
| Model Number | SM-004B |
| Brand Name | Safe•t mate ® |
| Description | Personal Fall Monitor |
| Manufacturer | 210 Innovations, LLC 210 Leonard Drive Groton, CT 06340 860-445-0210 |
| Recommended Use | The Safe•t mate Personal Fall Monitor is ideal for long-term care residents confined to a wheelchair or bed who are at risk for falls and need a staff alert as an alternative to a restraint. |
| Not recommended for | Patients who need a restraint Patients who may become agitated by the feeling of the garment clip attached to their clothing. |
| Operation | Alarm box does not have an on/off switch. The alarm will sound at the moment the magnet is pulled away from the metal face plate and can only be silenced by re-applying the magnet. The Cord features a slide adjustment. There should be enough slack in the cord to allow for normal movement without accidentally releasing the magnet. |
| Specifications | Alarm box ABS Plastic with Silicone sleeve. Powered by 9 volt battery. 95 db horn volume. LED indicator normally off. Flashing = low battery Low power drain circuit. Recommended battery change = Monthly Magnet assembly 1.4 MGOE low energy rubber magnet with energy field reaching < 1" from magnet Nylon cord length adjustable from 18" – 35" |
| Care | Wipe unit with cloth and mild spray cleaner such as glass cleaner. Do not submerge. Replacement Magnet assembly - Part no. 004BMAG |