

# Safe•t mate

## Product use and specification sheet

Model Number	<b>SM-007</b>
Brand Name	Safe•t mate ®
Description	Solid Drop Seat w Integrated alarm
Manufacturer	210 Innovations, LLC 210 Leonard Drive Groton, CT 06340 860.445.0210
Recommended Use	The Safe•t mate Solid Drop Seat w Integrated alarm is recommended for long-term care residents needing an alternative to a restraint and who also require a solid seat as a foundation for a custom seating cushion. Also recommended for residents that displace or tamper with typical alarm sensor pads. Should always be used with a seat cushion.
Not recommended for	Patients who need a restraint
Operation	Alarm box features on/off switch. With alarm switched on, at the moment weight is lifted from the wheelchair seat, the alarm sounds to alert patient and staff of the immediate fall risk. Once the resident returns to a seated position, the alarm will automatically reset.
Specifications	<b>Seat Deck</b> Seat deck is made from 3/8" thick PVC plastic. It measures 14" wide by 15.5" deep. Seat deck is suspended by 1.5" wide heavy duty polypropylene webbing w cam style buckles. Weight capacity: 0 lbs. Min. – 250 lbs. Max. Fits chairs 14.5" wide – 20" wide <b>Alarm box</b> ABS Plastic with Silicone sleeve. Powered by 9 volt battery. On/Off Switch. 95 db horn volume. 3.5mm audio jack. LED indicator. Low battery warning. Accidental cord removal warning. Low power drain circuit.
Care	Seat deck can remain on wheelchair during washing. Alarm box should be detached and wiped with cloth using mild detergent.