

Safe•t mate

Product use and specification sheet

Model Number	SM-009
Brand Name	Safe•t mate ®
Description	Under-seat Fall Monitoring System
Manufacturer	210 Innovations, LLC 210 Leonard Drive Groton, CT 06340 860.445.0210
Recommended Use	The Safe•t mate Under-seat Fall Monitoring System is ideal for long-term care residents needing an alternative to a restraint. Because the sensing unit is located beneath the wheelchair seat, this system is recommended for patients who tend to remove, soil or move out of position, above the seat sensor mat systems
Not recommended for	Patients who need a restraint
Operation	Alarm box features on/off switch. With alarm switched on, at the moment weight is lifted from seat, the alarm sounds to alert patient and staff of the immediate fall risk. Once safely seated, the alarm will automatically reset. Wheelchair can be fully collapsed for storage with unit installed
Specifications	Under-seat sensing unit Polycarbonate plastic. Spring pressure activated switch unit that attaches to separate alarm box. Attached to wheelchair w included hook and loop straps. Alarm box ABS Plastic with Silicone sleeve. Powered by 9 volt battery. On/Off Switch. 95 db horn volume. 3.5mm audio jack. LED indicator. Low battery warning. Accidental cord removal warning. Low power drain circuit.
Care	Alarm unit must be disconnected and removed prior to wheelchair washing Under-seat unit can remain installed during wheelchair washing Battery life will depend on usage. At a minimum, replace battery monthly.