

Safe•t mate

Product use and specification sheet

Model Number	SM-005, SM-005L, SM-005XL
Brand Name	Safe•t mate ®
Description	Hook & Loop Seatbelt w Alarm
Manufacturer	210 Innovations, LLC 210 Leonard Drive Groton, CT 06340 860.445.0210
Recommended Use	The Safe•t mate alarmed hook and loop seatbelt is ideal for long-term care residents needing an alternative to a restraint. The belt features a bright red pull strap that is easy for “self-releasing” type patients to locate. The closures is hook and loop so it can be released with a moderate amount of strength (less than 2 lbs. pull) and does not require the ability to operate a car type seatbelt button.
Not recommended for	Patients who need a restraint Patients who are not able to self release the belt Patients who are more likely to attempt to slide out of a closed belt rather than release it. Patients who become agitated (due to the belt application) to a degree that it causes safety risks that outweigh the intended benefit of the appliance.
Operation	Once the belt is installed, it should be adjusted for the individual user. It should not be overly tight causing discomfort nor should it be loose enough to allow the user to slide down in the wheelchair seat. Alarm box features on/off switch. With alarm switched on, at the moment the belt is separated, the alarm sounds to alert patient and staff of the immediate fall risk. Once the belt is re-fastened, the alarm will automatically reset.
Specifications	Belt SM-005 adjusts form 36”-46” total length SM-005L adjusts from 46”-58” total length Material is 2” wide nylon. Attachment point is brass grommets Recommended attachment method is direct to frame using existing seat back screws. Heavy duty tie wraps (2 rated AT 120 lbs ea.) supplied as alternate means of attachment. Buckle is polypropylene. Does not contain Latex Alarm box ABS Plastic with Silicone sleeve. Powered by 9 volt battery. On/Off Switch. 95 db horn volume. 3.5mm audio jack. LED indicator. Low battery warning. Accidental cord removal warning. Low power drain circuit.
Care	Belt can be machine washed. Mate two sides during washing to avoid hook and loop attracting other fabric threads. Alarm box should be wiped using cloth and mild detergent.